


Kangaroo Joey™ Enteral Feeding & Flush Pump

Loading & Rinsing of Feeding Pump

Do not use feeding sets for longer than 24 hours.

Proper loading & unloading of a feeding set

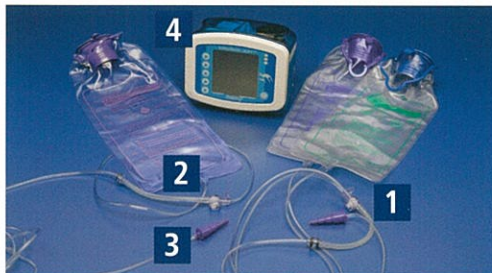

1) Feed & flush valve assembly, 2) Feed only valve assembly, 3) Distal end of feeding set, 4) Pump



1) Grasp thumb tab on valve and insert into left pocket, ensure valve is fully seated. Thumb tab should align with slot in blue door.




2) Grasp black ring retainer, gently wrap tubing around rotor, and insert retainer directly into right pocket. **Do not overstretch tubing.**



3) To properly unload a pump set, power the pump off and wait 10 seconds before removing the feeding set.

Proper rinsing of the feeding set

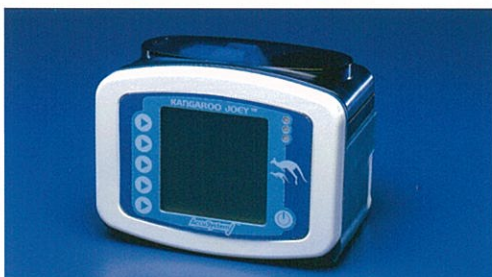

1) Disconnect feeding set from patient before rinsing. The feeding set must remain loaded during the rinsing procedure.



2) **CAUTION** – if set is removed from the pump, it may damage the set and prevent use and will negatively effect the system accuracy.



3) Empty contents of feed bag.



4) Fill the feed bag with water to approximately half of the total capacity. Properly seal the bag and shake for 5 to 10 seconds.



5) Empty the feed bag and repeat the last step until the bag contains clean water.



6) Power the pump on. Select "Keep Settings" to continue with previous setting or "Clear Settings" to start with programming new pump settings.



7) Press the "Prime Pump" button to enter the "Prime Pump" screen.



8) Press and hold the "Hold to Prime" button until only clean water flows through the tubing. Once the rotor starts this takes between 1 and 2 minutes.



9) Empty bag of water. Prime to remove any remaining water out of the feeding set.



10) **CAUTION** – To achieve proper accuracy the fluid level in the feeding set bag must be 6" above the top of the feeding pump, when initiating the feeding cycle.



COVIDIEN, COVIDIEN with Logo, Covidien logo and Kangaroo Joey are U.S. and/or internationally registered trademarks of Covidien AG.

© 2009 Covidien

H6211-11M-1209

15 HAMPSHIRE STREET
MANSFIELD, MA
02048

800-962-9888
508-261-8000

WWW.COVIDIEN.COM